

The **Peg Board System (PEGN613)** is an advanced tool designed to provide optimal stability and positioning for patients during surgeries and medical examinations. Specifically developed for lateral positioning, it allows for precise alignment and secure support, ensuring unrestricted access to targeted areas of the body. This system is essential for surgical teams requiring efficient and stable positioning, while minimizing patient discomfort and maintaining ease of access throughout the procedure.

Key Features


- Constructed from high-density polyethylene (HDPE), a durable material known for its excellent tensile strength and impact resistance, ensuring its durability and longevity in medical environments.
- The support posts come in three sizes – 8", 12", and 16" – to accommodate different patient sizes and procedural demands.
- The set includes four posts of each size, offering flexibility in placement.
- Included Components:
 - 1 x Peg Board
 - 1 x Gel Mat
 - 4 x 8" Posts
 - 4 x 12" Posts
 - 4 x 16" Posts
 - 4 x Table Straps

EVG

...

Who we are

Eighth Visionary Group (EVG) is a leading medical device supplier and distributor, driven by years of rigorous research and development within the operating room environment. Guided by the expertise of surgeons seeking enhanced solutions beyond those offered by existing medical device suppliers, EVG has developed locally designed products tailored to meet evolving clinical needs. As surgical techniques and approaches continue to advance, equipment must follow.

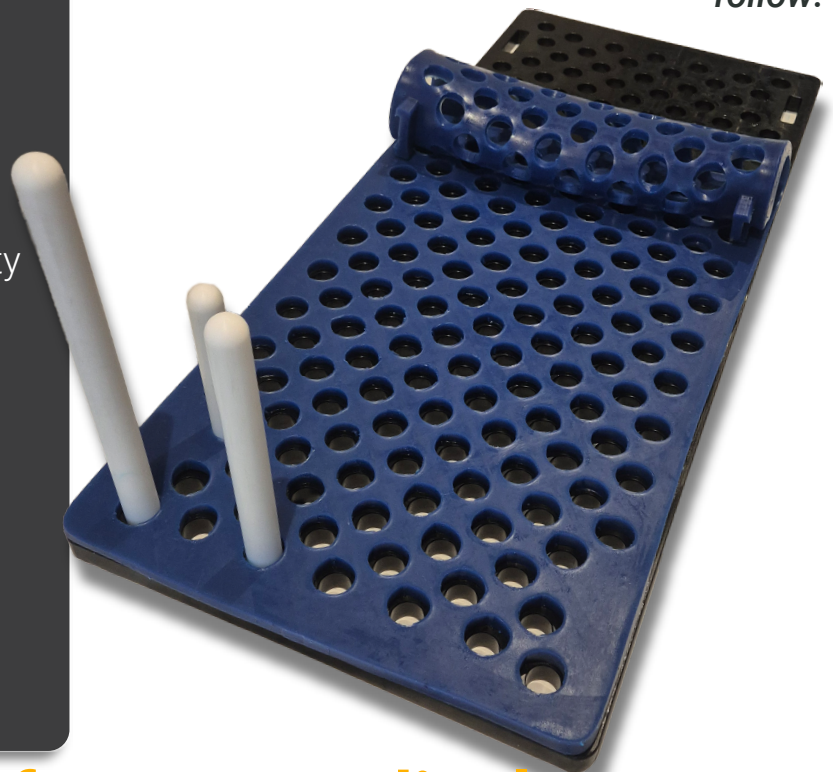


Email us at info@evgmedical.com

Directions of use

Note: When using the PEGN 613 two individuals will be required for stabilising the patient safely.

All patient positioning requirements are to be carried out under the local sites Manual Handling and Occupational Health and Safety Regulations. All positioning requirements are to be used under the supervision of the appropriate clinical team member(s) and under direction of the surgical team.

1. Remove the mattress from the operating table and securely attach the Peg Board to the table rails using the provided straps.
2. Place the gel mat over the Peg Board for added comfort and pressure relief.
3. Carefully transfer the patient onto the Peg Board and position them laterally as needed.
4. Position the posts to stabilize the patient. Depending on the area of access, use small, medium, or large posts to ensure the patient is securely positioned.
5. To minimize the risk of pressure sores, ensure that foam covers are used on the posts. Additionally, gel pads can be used for added comfort and pressure distribution as required.

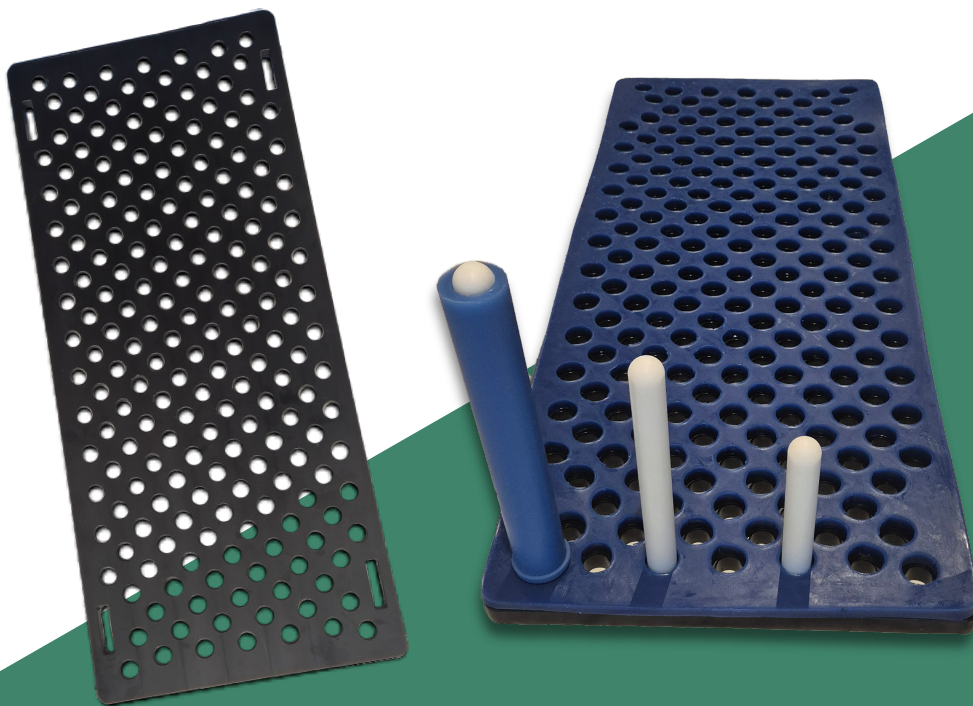

Email us at info@evgmedical.com today!

PEGN 613 products are engineered and manufactured to the highest standards. Each product is thoroughly tested before use and is guaranteed to be free from defects at the time of purchase. If a product fails due to a defect within one year of purchase, it will be either repaired or replaced. Our warranty does not cover damage caused by misuse or normal wear and tear.

